



duraclean

CONCENTRATED AUTOCLAVE CLEANER

Instructions For Use

CLEAN EVERY 25 CYCLES TO ENHANCE THE OPERATION AND PROLONG THE LIFE OF YOUR AUTOCLAVE

DURACLEAN is very beneficial for all autoclaves. Verify cleaning frequency in your autoclave's manual. When used as directed it not only cleans the chamber, steam lines and reservoir, but also helps clear up foul odors. DURACLEAN safely removes grease and grime using a mild, slightly alkaline liquid compound.

IMPORTANT: DO NOT STERILIZE INSTRUMENTS WHILE CLEANING AUTOCLAVE.

INSTRUCTIONS FOR USE:

1. Mix 1/2 gallon of distilled water with amount of cleaner specified on label. One quart bottle does five cleanings. Larger autoclaves may require more solution.
2. Drain old water from reservoir. Add cleaning solution from Step 1.
3. Run a normal cycle with no instruments.
4. Drain all cleaning solution from the reservoir. Add 1/2 gallon of clean distilled water for rinse cycle. Run a cycle with no instruments.
5. Drain reservoir and wipe out chamber, and clean debris off chamber rim and door gasket.
6. Refill reservoir with distilled water and autoclave is ready for use.



39 Western Highway, West Nyack, NY 10994

Technical Help-Line: 877-561-0500

Chemical Emergency: 800-535-5053