

Authorized Distributor 39 Western Hwy., West Nyack, NY 10994 www.duralinesystems.com sales@duralinesystems.com 877-561-0500



# **V10** AIR PLASMA STERILIZER

Redefining Veterinary Instrument Sterilization



## STERILIZE CRITICAL INSTRUMENTS

- ✓ surgical electronics
  - 🗸 lumen
  - ✓ optics

CERTIFIED

#### REDUCE OPERATING COSTS

- no installation cost
- ✓ low cost per cycle
- ✓ low maintenance


# plasmabionics.com

## ELIMINATE HARMFUL CHEMICALS

no chemical consumables
environmentally friendly
zero emissions

#### **INTRODUCING AIR PLASMA STERILIZATION®**

The VIO Air Plasma Sterilizer uses just air and electricity to sterilize critical veterinary instruments. The proprietary Air Plasma Sterilization<sup>®</sup> process produces multiple reactive oxygen and nitrogen species from air that rapidly destroy microorganisms on instrument surfaces. The sterilant gases are converted back into harmless air once sterilization is complete.

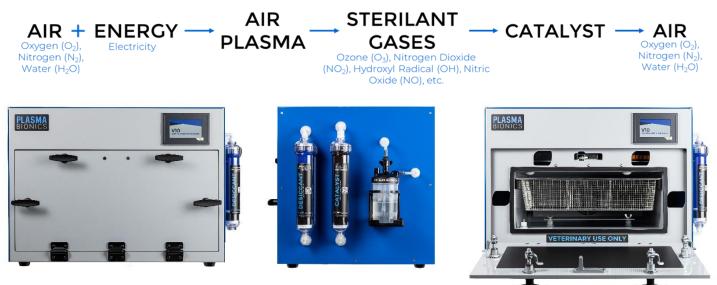

#### THE IDEAL ALTERNATIVE TO ETHYLENE OXIDE

Safely and easily sterilize critical instruments at least 3 times faster than ethylene oxide, increasing productivity and throughput. The Air Plasma Sterilization® process is compatible with surgical electronics and optics and may extend the usable life of sensitive veterinary instruments. By eliminating the need for high temperature, vacuum, high pressure, and harmful chemicals, the V10 Air Plasma Sterilizer provides a safe, intuitive, and environmentally friendly solution for veterinary instrument sterilization.

	Low temperature	Installation costs	Chemical consumables	Operating pressure	Cost per cycle	Sterilization cycle time
Air Plasma Sterilization®	yes	no	no	atmospheric	\$3-\$5	4 hours
Ethylene Oxide	yes	yes	yes	vacuum	\$20+	12-24 hours


# **TECHNICAL SPECIFICATIONS**

Exterior dimensions	28.75 x 19.25 x 18.75 in		
Sterilization tray dimensions	16 x 10 x 4 in		
Sterilization tray volume	10.5 L (2.8 gal)		
Weight	110 lbs (49.9 kg)		
Operating temperature	5-40 °C (41-104 °F)		
Storage temperature	-25-60 °C (-13-140 °F)		
Electrical requirements	110-120 VAC, 50/60 Hz, 6.5 A		
Preconditioning time	15-20 min		
Sterilizing time	3 hrs		
Ventilating time	45 min		

plasmabionics.com

CERTIFIED



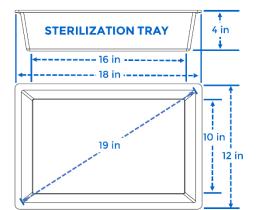

VIO Air Plasma Sterilizer Brochure\_Rev\_0822

Authorized Distributor 39 Western Hwy., West Nyack, NY 10994 www.duralinesystems.com

sales@duralinesystems.com 877-561-0500